

Vial2Bag


Instructions for Use

Before handling the components, wash your hands with soap and hot water.

Place the package on a clean, flat surface.



Step 1

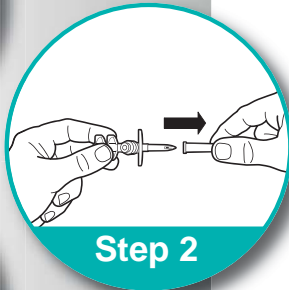


Step 2




Step 3



Step 4

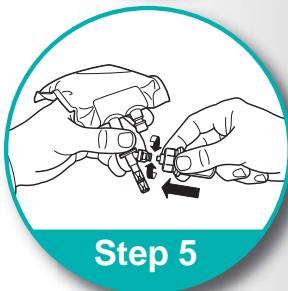


Step 5

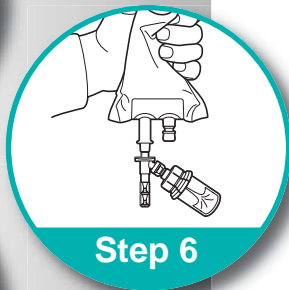

Step 6




Step 7

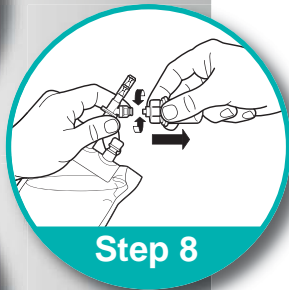

Step 8



Step 9

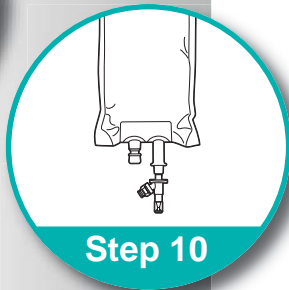

Step 10

Step 1: Remove the cover from the Vial2Bag package.
Remove the adapter from the blister pack.

Step 2: Remove the spike cover from the Vial2Bag and discard the cover.

Step 3: Insert the Vial2Bag spike into the IV port of the bag.

Step 4: Remove the Flip-Off® buttons from the lyophilized drug product vial.

Disinfect the injection site with an alcohol pad.

Seat the blue end of the device on the vial, pushing down until the spike penetrates the rubber stopper and the device snaps in place.



Step 5: Connect the vial to the luer port of the IV bag.

Step 6: With the vial lower than the bag, squeeze the bag to transfer the liquid from the bag into the vial to reconstitute the drug.

Step 7: Invert the vial and squeeze air from the bag into the vial; the reconstituted drug will flow into the bag.

Step 8: Remove the empty vial, or the vial can remain in place as a reminder that medication has been administered.

Step 9: Open the IV port on the Vial2Bag system.

Step 10: Remove the protector from the IV set spike and insert the spike into the IV port of the Vial2Bag system as you normally would.

The drug is now ready for administration.